

Empire Health Services - DPOM

PM Template Detail

Print Date: 6/24/2008 8:41:18

PM #:	15603	Annual PM for Bed, Hill Rom Advance 2000
--------------	--------------	---

Account: Obstetrics	Last Complete Date: 3/19/2008 7:48:46
Type: Active	Last Generated Date:
Skill: POM	Next Due Date: 2/1/2009
Priority: Routine/Priority	Allow Repeats? True
Frequency: Annual	WO Status: ACTIVE
Shop: Facilities Operations Mgmt	WO Sub-Stat: ISSUED, BEING WORKED ON

Instructions:

Asset Information

Asset #	Description	Manufacturer	Model	Serial #
Site	Building	Location		
D304051	Bed, Hill Rom Advance 2000	HILL ROM INC A	Advance 2000	1130-60A60-A
	Deaconess Medical	2	2022	

Assignments

Resource #	Resource Name	Resource Phone
Repairman	RepairmanD, Rotating	

Tasks

Task # 5643 **ANNUAL BED PM**

Est Hrs: 2 **Shutdown Required** No

Instructions:

- Run all moving parts to full extent making sure there is no binding, excessive noise or pinching of control or power cords.

- Lubricate all moving/sliding and hinged parts with light grease.

- Check electrical connections, cord and plug for tightness, condition and signs of fraying. Ensure that electrical cords are not excessively long, creating tripping hazards and making them subject to pinching by the bed mechanism.

- Check bed wheels for free movement and caster operation. Check brakes for holding. Remove lint and dirt and lubricate according to manufacturer's instruction.

- Perform electrical safety check. Check chassis to ground impedance (0.5 ohms max). Check leakage current in microamps (300ua max).

- Check drive motors for overheating or binding. Lubricate as required. Remove mattress, tighten nuts and bolts as required. Check for scrapes and scratches, repair as necessary.

- Inspect motor mounts and gear boxes. Caution: Remove from power source before examining any components, possible pinch or crushing points.